

Drug Cross Reactivity Table

| COMPOUND - Generic Name | COMPOUND - Trade Name | RESULTS |
|---|--------------------------------|--|
| Acetaminophen (See also Paracetamol) | Acephen | Non-reactive |
| Acetaminophen (See also Paracetamol) | Aceta | Non-reactive |
| Hydrocortisone | Actinac | Non-reactive |
| Nifedipine | Adalat | Non-reactive |
| Methylenedioxymethamphetamine (MDMA) | Adam (slang) | Positive for Methamphetamine (mAMP) and Ecstasy (MDMA) |
| Dexamphetamine Sulphate | Adderall | Positive for Amphetamine (AMP) |
| Dexamphetamine Sulphate | Adderall XR | Positive for Amphetamine (AMP) |
| Beclometasone | AeroBec | Non-reactive |
| Beclometasone | AeroBec Forte | Non-reactive |
| Salbutamol | Aerocrom | Non-reactive |
| Salbutamol | Aerolin | Non-reactive |
| Oxymetazoline | Afrin | Non-reactive |
| Pseudoephedrine | Afrinol | Non-reactive |
| Salbutamol | Airomir | Non-reactive |
| Naproxen | Aleve | Non-reactive |
| Aluminum Hydroxide | Alisone | Non-reactive |
| Promethazine | Allerfen | Positive for Tricyclic Antidepressants (TCA) |
| Hydrocortisone | Alphaderm | Non-reactive |
| Hydrocortisone | Alphosyl- HC | Non-reactive |
| Aluminum Hydroxide | Alu-Cap | Non-reactive |
| Amoxicillin or Amoxycillin | Amoram | Non-reactive |
| Amoxicillin or Amoxycillin | Amoxil | Non-reactive |
| Pericyazine | Amplan | Non-reactive |
| Amobarbital | Amytal | Positive for Barbiturates (BAR) |
| Metronidazole | Anabact | Non-reactive |
| Acetylsalicylic acid | Anadin | Non-reactive |
| Acetylsalicylic acid | Anasin | Non-reactive |
| Testosterone | Andropatch | Non-reactive |
| Promethazine | Anergan | Positive for Tricyclic Antidepressants (TCA) |
| Aspirin | Angettes | Non-reactive |
| Isosorbide Dinitrate | Angitak | Non-reactive |
| Aluminum Chloride Hexahydrate | Anhydrol Forte | Non-reactive |
| Naltrexone | Antaxone | Non-reactive |
| Promethazine | Antinaus | Positive for Tricyclic Antidepressants (TCA) |
| Hydrocortisone | Anugesic- HC | Non-reactive |
| Hydrocortisone | Anusol-HC | Non-reactive |
| Medazepam | Anxitol | Positive for Benzodiazepines (BZO) |
| Pericyazine | Aolept | Non-reactive |
| Acetaminophen (See also Paracetamol) | Apacet | Non-reactive |
| Pericyazine | Apamin | Non-reactive |
| Bendrofluazide | Aprinox | Non-reactive |
| Promethazine | Aprobit | Positive for Tricyclic Antidepressants (TCA) |
| Chloroquine | Aralen | Positive for Methamphetamine (mAMP) |
| Erythromycin | Arpimycin | Non-reactive |
| Aspirin | ASA | Non-reactive |
| Aspirin | Asasantin | Non-reactive |
| Dimethicone | Asilone | Non-reactive |
| Beclometasone | Asmabec | Non-reactive |
| Salbutamol Non-reactive | Asmasal | Non-reactive |
| Acetylsalicylic acid Non-reactive | Aspirin | Non-reactive |
| Morphine | Astramorph | Positive for Opiates (OPI/MOR) |
| Hydroxyzine Hydrochloride Non-reactive | Atarax | Non-reactive |
| Lorazepam | Ativan | Positive for Benzodiazepines (BZO) |
| Amoxicillin or Amoxycillin Non-reactive | Augmentin | Non-reactive |
| Co-Amoxiclav Non-reactive | Augmentin | Non-reactive |
| Co-Amoxiclav | Augmentin Duo | Non-reactive |
| Promethazine | Avomine | Positive for Tricyclic Antidepressants (TCA) |
| Chlorhexidine Gluconate Non-reactive | Bactrigras | Non-reactive |
| Aspirin Non-reactive | Bayer Aspirin | Non-reactive |
| Promethazine | Baymethzine | Positive for Tricyclic Antidepressants (TCA) |
| Beclometasone Non-reactive | Beclazone | Non-reactive |
| Beclometasone Non-reactive | Becloforte | Non-reactive |
| Beclometasone Non-reactive | Becodisks | Non-reactive |
| Beclometasone Non-reactive | Beconase | Non-reactive |
| Beclometasone | Becotide | Non-reactive |
| Diphenhydramine | Benadryl | Non-reactive |
| Erythromycin | Benzymycin | Non-reactive |
| Atenolol | Beta-adlat | Non-reactive |
| Propranolol | Beta-Progane | Non-reactive |
| Norethindrone | BiNovum | Non-reactive |
| Benzalkonium | Bradosol | Non-reactive |
| Norethindrone | Brevinor | Non-reactive |
| Delorazepam | Briantum | Positive for Benzodiazepines (BZO) |
| Lotexidine | Britoflex | Non-reactive |
| Ibuprofen | Brufen | Non-reactive |
| Acetylsalicylic acid | Bufferin | Non-reactive |
| Hyoscine butylbromide | Busocopan | Non-reactive |
| Butabarbital | Butisol | Positive for Barbiturates (BAR) |
| Hydrocortisone | Calmurid-HC | Non-reactive |
| Canestan | Canestan ear/skin/vaginitis/HC | Non-reactive |
| Hydrocortisone | Canesten- HC | Non-reactive |
| Acetylsalicylic acid | Caprin | Non-reactive |
| Aspirin | Caprin | Non-reactive |
| Lisinopril HCl | Carace | Non-reactive |
| Isosorbide Dinitrate | Cedocard | Non-reactive |
| Prazepam | Centrax | Positive for Benzodiazepines (BZO) |
| Cephalexin | Ceporex | Non-reactive |
| Chlorhexidine Gluconate | Chlorohex | Non-reactive |
| Gentamicin Sulphate | Cidomycin | Non-reactive |
| Citalopram | Cipramil | Non-reactive |
| Loratadine | Claritin | Non-reactive |
| Norethindrone | Climagest | Non-reactive |
| Norethindrone | Climesse | Non-reactive |
| Clonazepam | Clonopin | Positive for Benzodiazepines (BZO) |
| Paracetamol (Acetaminophen) /Codeine Preparations | Co-codamol | Positive for Opiates (OPI/MOR) |

Drug Cross Reactivity Table

| COMPOUND - Generic Name | COMPOUND - Trade Name | RESULTS |
|---|------------------------|---|
| Ibuprofen | Codafen | Non-reactive |
| Paracetamol (Acetaminophen) /Codeine Preparations | Codafen | Positive for Opiates (OPI/MOR) |
| Codeine Phosphate | Codafen Continus | Positive for Opiates (OPI/MOR) |
| Codeine Phosphate | Codeine Linctus | Positive for Opiates (OPI/MOR) |
| Paracetamol (Acetaminophen) /Codeine Preparations | Co-dydramol | Positive for Opiates (OPI/MOR) |
| Flucloxacillin | Co-fluampicil | Non-reactive |
| Bendrofluzide | Cogaretic | Non-reactive |
| Hydrocortisone | Coloam | Non-reactive |
| Penicillin | Combicillin | Non-reactive |
| Salbutamol | Combivent | Non-reactive |
| Naproxen | Condrotec | Non-reactive |
| Dextropropoxyphene | Co-proximal | Positive for Propoxyphene (PPX) |
| Chlorhexidine Gluconate | Cordsodyl | Non-reactive |
| Nadolol | Corgard | Non-reactive |
| Nadolol | Corgaretic | Non-reactive |
| Nadolol | Corgartic | Non-reactive |
| Atenolol Sodium | Co-tenidone | Non-reactive |
| Benzylpenicillin | Crystapen | Non-reactive |
| Chlorhexidine Gluconate | CX Powder | Non-reactive |
| Morphine | Cyclimorph | Positive for Opiates (OPI/MOR) |
| Hydrocortisone | Daktacort | Non-reactive |
| Acetaminophen (See also Paracetamol) | Dapacen | Non-reactive |
| Dextropropoxyphene | Darvocet | Positive for Propoxyphene (PPX) |
| Dextropropoxyphene | Darvon | Positive for Propoxyphene (PPX) |
| Dexamethasone | Decadron | Non-reactive |
| Prednisolone | Delta-cortef | Non-reactive |
| Meperidine | Demerol | Non-reactive |
| Prazepam | Demetrin | Positive for Benzodiazepines (BZO) |
| Sodium Valproate | Depakene | Non-reactive |
| Sodium Valproate | Depakote | Non-reactive |
| Flupentixol Decanoate | Depixol | Non-reactive |
| Chlorhexidine Gluconate | Dermol | Non-reactive |
| d-Methamphetamine HCL | Desoxyn | Positive for Methamphetamine (mAMP) |
| Trazodone | Desyrel | Non-reactive |
| Dexamethasone | Dexa-Rhinaspray Duo | Non-reactive |
| Dexamphetamine Sulphate | Dexedrine | Positive for Amphetamine (AMP) |
| Dihydrocodeine | DHC Continus | Positive for Opiates (OPI/MOR) |
| Diazepam | Diazemuls | Positive for Benzodiazepines (BZO) |
| Diclofenac Sodium | Dicloflex | Non-reactive |
| Diclofenac Sodium | Diclomax | Non-reactive |
| Benzylamine HCl | Diffiam | Non-reactive |
| Phenytoin | Dilantin | Potential Positive for Barbiturates (BAR) Urinary metabolite(s) only; parent compound is non-reactive |
| Hydromorphone | Dilaudid | Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY) |
| Brompheniramine | Dimetapp | Non-reactive |
| Brompheniramine | Dimotane | Non-reactive |
| Brompheniramine | Dimotapp | Non-reactive |
| Koalin and Morphine Mixture | Diocalm | Positive for Opiates (OPI/MOR) |
| Hydrocortisone | Dioderm | Non-reactive |
| Promethazine | Diprozin | Positive for Tricyclic Antidepressants (TCA) |
| Promethazine | Diprozin | Non-reactive |
| Orphenadrine | Disipil | Non-reactive |
| Acetylsalicylic acid | Disprin | Non-reactive |
| Methadone Hydrochloride | Dolophine | Positive for Methadone (MTD) |
| Phenobarbital | Donnatal | Positive for Barbiturates (BAR) |
| Aluminum Chloride Hexahydrate | Drichlor | Non-reactive |
| Salbutamol | Duovent | Non-reactive |
| Lactulose | Duphalec | Non-reactive |
| Fentanyl | Duragesic | Non-reactive |
| Morphine | Duramorph | Positive for Opiates (OPI/MOR) |
| Cimetidine | Dyspamet | Non-reactive |
| Methylenedioxyamphetamine (MDMA) | E (slang) | Positive for Methamphetamine (mAMP) and Ecstasy (MDMA) |
| Hydrocortisone | Econacort | Non-reactive |
| Prednisolone | Econopred | Non-reactive |
| Acetylsalicylic acid | Ecotrin | Non-reactive |
| Methylenedioxyamphetamine (MDMA) | Ecstasy (slang) | Positive for Methamphetamine (mAMP) and Ecstasy (MDMA) |
| Hydrocortisone | Efcortesel | Non-reactive |
| Venlafaxine | Effexor | Potential Positive for Phencyclidine (PCP) |
| Venlafaxine | Effexor XL | Potential Positive for Phencyclidine (PCP) |
| Hydrocortisone | Efortelan | Non-reactive |
| Amitriptyline | Elavil | Positive for Tricyclic Antidepressants (TCA) |
| Norethindrone | Elleste Duet | Non-reactive |
| Thyroxine Sodium | Eltroxin | Non-reactive |
| Metronidazole | Elyzol | Non-reactive |
| Acetylsalicylic acid | Empirin | Non-reactive |
| Koalin and Morphine Mixture | Entersan | Positive for Opiates (OPI/MOR) |
| Phenytoin | Epanutin | Potential Positive for Barbiturates (BAR) Urinary metabolite(s) only; parent compound is non-reactive |
| Sodium Valproate | Epilim | Non-reactive |
| Phenytoin | Epitard | Potential Positive for Barbiturates (BAR) Urinary metabolite(s) only; parent compound is non-reactive |
| Erythromycin | Erycane | Non-reactive |
| Erythromycin | Erymax | Non-reactive |
| Erythromycin | Erythrocin | Non-reactive |
| Erythromycin | Erythroped | Non-reactive |
| Norethindrone | Estracombi | Non-reactive |
| Ethylmorphine | Ethylmorphine | Positive for Opiates (OPI/MOR) |
| Hydrocortisone | Eurax-HC | Non-reactive |
| Methylenedioxyamphetamine (MDA) | Eve (slang) | Positive for Amphetamine (AMP) and Ecstasy (MDMA) |
| Norethindrone | Evorel | Non-reactive |
| Aspirin | Excedrin | Non-reactive |
| Acetylsalicylic acid | Excedrin (combination) | Non-reactive |
| Acetaminophen (See also Paracetamol) | Excedrin (combination) | Non-reactive |
| Dimethicone | Fancorsil | Non-reactive |
| Ibuprofen | Fenbid | Non-reactive |
| Promethazine | Fenergan | Positive for Tricyclic Antidepressants (TCA) |
| Acetaminophen (See also Paracetamol) | Feverall | Non-reactive |
| Beclometasone | Fillair | Non-reactive |

Drug Cross Reactivity Table

| COMPOUND - Generic Name | COMPOUND - Trade Name | RESULTS |
|---|-----------------------|---|
| Butalbital | Fioricet | Positive for Barbiturates (BAR) |
| Butalbital | Fiorinal | Positive for Barbiturates (BAR) |
| Metronidazole | Flagyl | Non-reactive |
| Indomethacin | Flexin continus | Non-reactive |
| Flucloxacillin | Floxapen | Non-reactive |
| Flupentixol Decanoate | Fluanxol | Non-reactive |
| Paracetamol (Acetaminophen) | Fortagesic | Non-reactive |
| Clobazam | Frisium | Positive for Benzodiazepines (BZO) |
| Fusidic Acid (Sodium Fusidate) | Fucibet | Non-reactive |
| Fusidic Acid (Sodium Fusidate) | Fucidin | Non-reactive |
| Fusidic Acid (Sodium Fusidate) | Fucithalmic | Non-reactive |
| Codeine Phosphate | Galcodine | Positive for Opiates (OPI/MOR) |
| Pholcodine | Galenphol | Positive for Opiates (OPI/MOR) |
| Chlorpheniramine Maleate | Galpseud | Non-reactive |
| Lofepramine | Gamanil | Positive for Tricyclic Antidepressants (TCA) |
| Aluminum Hydroxide | Gastrocote | Non-reactive |
| Magnesium Alginate | Gaviscon | Non-reactive |
| Mannitol | Gaviscon | Non-reactive |
| Gentamicin Sulphate | Genticin | Non-reactive |
| Gentamicin Sulphate | Gentisone | Non-reactive |
| Hydrocortisone | Gregoderm | Non-reactive |
| Triazolam | Halcion | Positive for Benzodiazepines (BZO) |
| Chlorpheniramine Maleate | Haymine | Non-reactive |
| Amoxicillin or Amoxycillin | Heliclear | Non-reactive |
| Lansoprazole | Heliclear | Non-reactive |
| Chlorhexidine Gluconate | Hibicet | Non-reactive |
| Chlorhexidine Gluconate | Hibiscrub | Non-reactive |
| Chlorhexidine Gluconate | Hibisol | Non-reactive |
| Chlorhexidine Gluconate | Hibitane | Non-reactive |
| Hydrocortisone | Hydrocortistab | Non-reactive |
| Hydrocortisone | Hydrocortone | Non-reactive |
| Hydromorphone | Hydrostat | Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY) |
| Ibuprofen | Ibugel | Non-reactive |
| Ibuprofen | Ibuspray | Non-reactive |
| Erythromycin | Ilosone | Non-reactive |
| Loperamide HCl | Imodium | Non-reactive |
| Propranolol | Inderal | Non-reactive |
| Propranolol | Inderetic | Non-reactive |
| Bendrofluzide | Inderex | Non-reactive |
| Propranolol | Inderex | Non-reactive |
| Bendrofluzide | Indertic | Non-reactive |
| Indomethacin | Indocid | Non-reactive |
| Indomethacin | Indomod | Non-reactive |
| Prednisolone | Inflamase | Non-reactive |
| Isoniazide | INH | Non-reactive |
| Chlorhexidine Gluconate | Instillagel | Non-reactive |
| A Interferon Alfa | Intron | Non-reactive |
| Ammonia/Ipeacuanha | Ipecac | Non-reactive |
| Pericyazine | Iryakin | Non-reactive |
| Isosorbide Dinitrate | Isocard | Non-reactive |
| Isosorbide Dinitrate | Isoket | Non-reactive |
| Isosorbide Dinitrate | Isordil | Non-reactive |
| Erythromycin | Isotrexin | Non-reactive |
| Atenolol | Kalten | Non-reactive |
| Paracetamol (Acetaminophen) /Codeine Preparations | Kapake | Positive for Opiates (OPI/MOR) |
| Cephalexin | Keflex | Non-reactive |
| Norethindrone | Kliofem | Non-reactive |
| Norethindrone | Kliovance | Non-reactive |
| Clonazepam | Klonopin | Positive for Benzodiazepines (BZO) |
| Aluminum Hydroxide | Kolanticon | Non-reactive |
| Dicyclomine | Kolanticon | Non-reactive |
| Erythromycin | Lactobinate | Non-reactive |
| Lactulose | Lactugal | Non-reactive |
| Lamotrigine | Lamitalc | Positive for Phencyclidine (PCP) |
| Chlorpromazine | Largactil | Non-reactive |
| Amitriptyline | Lentizol | Positive for Tricyclic Antidepressants (TCA) |
| Medazepam | Lerisum | Positive for Benzodiazepines (BZO) |
| Bromazepam | Lexotan | Positive for Benzodiazepines (BZO) |
| Chlordiazepoxide | Librium | Positive for Benzodiazepines (BZO) |
| Atorvastatin | Lipitor | Non-reactive |
| Hydrocortisone-17-butyrate | Locoid | Non-reactive |
| Hydrocortisone-17-butyrate | Locoid C | Non-reactive |
| Norethindrone | Loestrin | Non-reactive |
| Lofepramine | Lomont | Positive for Tricyclic Antidepressants (TCA) |
| Co-Phenotrope (atropine/diphenoxylate) | Lomotil | Non-reactive |
| Diphenoxylate with Atropine | Lomotil | Non-reactive |
| Loperamide HCl | Loperagen | Non-reactive |
| Hydrocodone | Lorcet | Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY) |
| Hydrocodone | Lortab | Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY) |
| Methylenedioxyamphetamine (MDA) | Love Drug (slang) | Positive for Amphetamine (AMP) and Ecstasy (MDMA) |
| Phenobarbitone (see also Phenobarbital) | Luminal | Positive for Barbiturates (BAR) |
| Aluminum Hydroxide | Maalox | Non-reactive |
| Aluminum Hydroxide | Maalox TC | Non-reactive |
| Flucloxacillin | Magnapen | Non-reactive |
| Co-Fluampicil | Magnapen | Non-reactive |
| Warfarin Sodium | Marevan | Non-reactive |
| Dronabinol | Marinol | Positive for Cannabinoids (THC) Urinary metabolite(s) only; parent compound is non-reactive |
| Dexamethasone | Maxidex | Non-reactive |
| Dexamethasone | Maxidrol | Non-reactive |
| Medazepam | Medacepan | Positive for Benzodiazepines (BZO) |
| Diphenhydramine | Medinex | Non-reactive |
| Penicillin | Mefoxin | Non-reactive |
| Thioridazine Hydrochloride | Mellaril | Non-reactive |
| Dicyclomine | Merbentyl | Non-reactive |
| Methadone Hydrochloride | Methadose | Positive for Methadone (MTD) |

Drug Cross Reactivity Table

| COMPOUND - Generic Name | COMPOUND - Trade Name | RESULTS |
|--------------------------------------|-----------------------|---|
| d-Methamphetamine HCL | Methamprex | Positive for Methamphetamine (mAMP) |
| Promethazine | Methazine | Positive for Tricyclic Antidepressants (TCA) |
| d-Methamphetamine HCL | Methedrine | Positive for Methamphetamine (mAMP) |
| Metronidazole | Metrogel | Non-reactive |
| Metronidazole | Metrolyl | Non-reactive |
| Metronidazole | Metrotop | Non-reactive |
| Norethindrone | Micronor | Non-reactive |
| Codeine Phosphate | Migravele | Positive for Opiates (OPI/MOR) |
| Ilocream Hydrocortisone | Mildison | Non-reactive |
| Dexamethasone | Minims | Non-reactive |
| Nitrazepam | Mogadon | Positive for Benzodiazepines (BZO) |
| Trazodone | Molipaxin | Non-reactive |
| Morphine | Morcap | Positive for Opiates (OPI/MOR) |
| Sulfate Morphine | Morphine | Positive for Opiates (OPI/MOR) |
| Morphine | Morphine Sulfate | Positive for Opiates (OPI/MOR) |
| Diclofenac Sodium | Motifene | Non-reactive |
| Ibuprofen | Motrin | Non-reactive |
| Contin Morphine | MS | Positive for Opiates (OPI/MOR) |
| Morphine | MS Contin | Positive for Opiates (OPI/MOR) |
| Aluminum Hydroxide | Mucogel | Non-reactive |
| Ethambutol | Myambutol | Non-reactive |
| Naltrexone | Nalorex | Non-reactive |
| Naproxen | Napratec | Non-reactive |
| Naproxen | Naprosen | Non-reactive |
| Naproxen | Naprosyn | Non-reactive |
| Naloxone | Narcan | Non-reactive |
| Noscapine | Narcotin | Non-reactive |
| Chlorhexidine Gluconate | Naseptin | Non-reactive |
| Beclometasone | Nasobec | Non-reactive |
| Pericyazine | Nemactil | Non-reactive |
| Pentobarbital | Nembutal | Positive for Barbiturates (BAR) |
| Hydrocortisone | Neo-cortef | Non-reactive |
| Bendrofluazide | Neo-Naclax | Non-reactive |
| Bendrofluazide | Neo-Naclax-K | Non-reactive |
| Oxymetazoline | Neo-synephrine | Non-reactive |
| Pericyazine | Neulactil | Non-reactive |
| Metronidazole | Neuratop | Non-reactive |
| | Nexium | Positive for Cannabinoids (THC) Urinary metabolite(s) only; parent compound is non-reactive |
| Nicotine | Nicoderm | Non-reactive |
| Nicotine | Nicorette | Non-reactive |
| Nicotine | Nicotinell | Non-reactive |
| Nicotine | Nicotrol | Non-reactive |
| Nifedipine | Nifar | Non-reactive |
| Nifedipine | Nifecard | Non-reactive |
| Nifedipine | Nif-ten | Non-reactive |
| Nicotine | Niquitin | Non-reactive |
| Alphenol | No known trade names | Positive for Barbiturates (BAR) |
| Allobarbitol | No known trade names | Positive for Barbiturates (BAR) |
| d,l-Octopamine | No known trade names | Non-reactive |
| Medazepam | Nobritol | Positive for Benzodiazepines (BZO) |
| Medazepam | Nobrium | Positive for Benzodiazepines (BZO) |
| Lormetazepam | Noctamide | Positive for Benzodiazepines (BZO) |
| Orphenadrine | Norflex | Non-reactive |
| Orphenadrine | Norgesic | Non-reactive |
| Norethindrone | Noriday | Non-reactive |
| Loperamide HCl | Norimide | Non-reactive |
| Norethindrone | Norimin | Non-reactive |
| Norethindrone | Norinyl | Non-reactive |
| Norethindrone | Noristat | Non-reactive |
| Metronidazole | Noritate | Non-reactive |
| Norethindrone | Norlutin | Non-reactive |
| Procaine | Novocain | Positive for Opiates (OPI/MOR) |
| Oxymorphone | Numorphan | Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY) |
| Norethindrone | Nuvelle | Non-reactive |
| Naproxen | Nycopren | Non-reactive |
| Doxylamine | Nyquil | Positive for Methadone (MTD) |
| Hydrocortisone | Nystaform | Non-reactive |
| Chlorhexidine Gluconate | Nystaform | Non-reactive |
| Diphenhydramine | Nytol | Non-reactive |
| Koalin and Morphine Mixture | Opazimes | Positive for Opiates (OPI/MOR) |
| Morphine | Oramorph | Positive for Opiates (OPI/MOR) |
| Ketoprofen | Orudis | Non-reactive |
| Ketoprofen | Oruvail | Non-reactive |
| Dexamethasone | Otomize | Non-reactive |
| Antazoline Sulphate | Otrivine-Anistin | Non-reactive |
| Norethindrone | Ovysmen | Non-reactive |
| Oxazepam | Ox-pam | Positive for Benzodiazepines (BZO) |
| Oxycodone | OxyContin | Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY) |
| Acetaminophen (See also Paracetamol) | Panadol | Non-reactive |
| Diphenhydramine | Panadol Night | Non-reactive |
| Paracetamol (Acetaminophen) | Paradote | Non-reactive |
| Dihydrocodeine | Paramol | Positive for Opiates (OPI/MOR) |
| Pholcodine | Pavacol-D | Positive for Opiates (OPI/MOR) |
| Paroxetine | Paxil | Non-reactive |
| Prednisolone | Pediapred | Non-reactive |
| Codeine Phosphate | Pediatric BP | Positive for Opiates (OPI/MOR) |
| Ampicillin | Penbritin | Non-reactive |
| Oxycodone | Percolone | Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY) |
| Hydrocortisone | Perinal | Non-reactive |
| Meperidine | Pethidine | Non-reactive |
| Acetophenetidin | Phenacetin | Non-reactive |
| Promethazine | Phenergen | Positive for Tricyclic Antidepressants (TCA) |
| Promethazine | Phenergen | Non-reactive |
| Methadone Hydrochloride | Physetone | Positive for Methadone (MTD) |
| Piperacillin | Pipracil | Non-reactive |

Drug Cross Reactivity Table

| COMPOUND - Generic Name | COMPOUND - Trade Name | RESULTS |
|---|--------------------------------|---|
| Chlorpheniramine Maleate | Pirton | Non-reactive |
| Hydroxychloroquine | Plaquenil | Non-reactive |
| Ampicillin | Polyclin | Non-reactive |
| Mefenamic Acid | Ponstan | Non-reactive |
| Mefenamic Acid | Ponstel | Non-reactive |
| Ketoprofen | Powergel | Non-reactive |
| Prednisolone | Prelone | Non-reactive |
| Bendrofluazide | Prestin | Non-reactive |
| | Prilosec | Positive for Cannabinoids (THC) Urinary metabolite(s) only; parent compound is non-reactive |
| Ampicillin | Principen | Non-reactive |
| Nifedipine | Procardia | Non-reactive |
| Hydrocortisone | Proctofoam | Non-reactive |
| Ibuprofen | Proflex | Non-reactive |
| Pericyazine | Propetyl | Non-reactive |
| Estazolam | ProSom | Positive for Benzodiazepines (BZO) |
| Dothiepin | Prothiaden | Non-reactive |
| Pantoprazole | Protonix | Positive for Cannabinoids (THC) Urinary metabolite(s) only; parent compound is non-reactive |
| Fluoxetine | Prozac | Non-reactive |
| Pericyazine | Psycholept | Non-reactive |
| Ranitidine | Pylorid | Positive for Methamphetamine (mAMP) Urinary metabolite(s) only; parent compound is non-reactive |
| Hydroxide | Pyrogastone Aluminum | Non-reactive |
| Hydrocortisone | Quinocort | Non-reactive |
| Beclometasone | Qvar | Non-reactive |
| Metoclopramide | Reglan | Non-reactive |
| Dihydrocodeine | Remedeine | Positive for Opiates (OPI/MOR) |
| Dihydrocodeine | Remedeine Forte | Positive for Opiates (OPI/MOR) |
| Paracetamol (Acetaminophen) /Codeine Preparations | Remedine | Positive for Opiates (OPI/MOR) |
| Testosterone | Restandol | Non-reactive |
| Temazepam | Restoril | Positive for Benzodiazepines (BZO) |
| Clonazepam | Rivotril | Positive for Benzodiazepines (BZO) |
| Interferon Alfa | Roferon-A | Non-reactive |
| Flunitrazepam | Rohypnol | Positive for Benzodiazepines (BZO) |
| Morphine | Roxanol | Positive for Opiates (OPI/MOR) |
| Oxycodone | Roxicodone | Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY) |
| Metronidazole | Rozex | Non-reactive |
| Fluoxetine | Sarafem | Non-reactive |
| Hyoscine butylbromide | Scopoderm | Non-reactive |
| Secobarbital | Seconal | Positive for Barbiturates (BAR) |
| Azlocillin | Securopen | Non-reactive |
| Sennosides | Senokot | Non-reactive |
| Oxazepam | Serax | Positive for Benzodiazepines (BZO) |
| Chlorhexidine Gluconate | Serotulle | Non-reactive |
| Paroxetine | Seroxat | Non-reactive |
| Morphine | Severedol | Positive for Opiates (OPI/MOR) |
| Doxepin | Sinequan | Positive for Tricyclic Antidepressants (TCA) |
| Testosterone | slang name "anabolic steroids" | Non-reactive |
| Dexamethasone | Sofradex | Non-reactive |
| Paracetamol (Acetaminophen) /Codeine Preparations | Solpadol | Positive for Opiates (OPI/MOR) |
| Nitrazepam | Somnite | Positive for Benzodiazepines (BZO) |
| Butabarbital | Soneryl | Positive for Barbiturates (BAR) |
| Isosorbide Dinitrate | Sorbichew | Non-reactive |
| Isosorbide Dinitrate | Sorbid SA | Non-reactive |
| Isosorbide Dinitrate | Sorbitrate | Non-reactive |
| Alverine Citrate | Spasmonal | Non-reactive |
| Alverine Citrate | Spasmonal Fibre | Non-reactive |
| Chlorhexidine Gluconate | Steripod | Non-reactive |
| Diazepam | Stesolid | Positive for Benzodiazepines (BZO) |
| Erythromycin | Stiemycin | Non-reactive |
| Pholcodine | Strong BP | Positive for Opiates (OPI/MOR) |
| Buprenorphine | Suboxone | Positive for Buprenorphine (BUP) |
| Buprenorphine | Subutex | Positive for Buprenorphine (BUP) |
| Pseudoephedrine | Sudafed | Non-reactive |
| Testosterone | Sustanon | Non-reactive |
| Efavirenz | Sustiva | Positive for Cannabinoids (THC) Urinary metabolite(s) only; parent compound is non-reactive |
| Naproxen | Synflex | Non-reactive |
| Cimetidine | Tagamet | Non-reactive |
| Piperacillin | Tazobactam | Non-reactive |
| Carbamazepine | Tegretol | Non-reactive |
| Buprenorphine | Temgesic | Positive for Buprenorphine (BUP) |
| Acetaminophen (See also Paracetamol) | Tempra | Non-reactive |
| Atenolol | Tenben | Non-reactive |
| Bendrofluazide | Tenben | Non-reactive |
| Atenolol | Tenif | Non-reactive |
| Atenolol | Tenorect | Non-reactive |
| Atenolol | Tenoret 50 | Non-reactive |
| Atenolol | Tenormin | Non-reactive |
| Carbamazepine | Teril | Non-reactive |
| Hydrocortisone | Terra-cortil | Non-reactive |
| Oxytetracycline | Terra-Cortil | Non-reactive |
| Oxytetracycline | Terramycin | Non-reactive |
| Pholcodine | Thebacon | Positive for Opiates (OPI/MOR) |
| Erythromycin | Tilorith | Non-reactive |
| Hydrocortisone | Timodine | Non-reactive |
| Carbamazepine | Timonil | Non-reactive |
| Chlorhexidine Gluconate | Tisept | Non-reactive |
| Aluminum Hydroxide | Topal | Non-reactive |
| Tramadol | Tramake | Non-reactive |
| Chlorazepate | Tranxene | Positive for Benzodiazepines (BZO) |
| Co-Phenotrope (atropine/diphenoxylate) | Trepergen | Non-reactive |
| Naltrexone | Trexan | Non-reactive |
| Trazodone | Trialodine | Non-reactive |
| Oxytetracycline | Trimovate | Non-reactive |
| Norethindrone | TriNovum | Non-reactive |
| Amitriptyline | Triptafen | Positive for Tricyclic Antidepressants (TCA) |
| Amitriptyline | Triptafen-M | Positive for Tricyclic Antidepressants (TCA) |

Drug Cross Reactivity Table

| COMPOUND - Generic Name | COMPOUND - Trade Name | RESULTS |
|--|----------------------------|---|
| Norethindrone | Trisquens | Non-reactive |
| Amitriptyline | Tryptizol | Positive for Tricyclic Antidepressants (TCA) |
| Amobarbital | Tuinal | Positive for Barbiturates (BAR) |
| Acetaminophen (See also Paracetamol) | Tylenol | Non-reactive |
| Paracetamol (Acetaminophen) /Codeine Preparations | Tylenol 3 | Positive for Opiates (OPI/MOR) |
| Pseudoephedrine | Tylenol Cold (combination) | Non-reactive |
| Acetaminophen with Codeine (see also Paracetamol with codeine) | Tylenol with codeine | Positive for Opiates (OPI/MOR) |
| Paracetamol (Acetaminophen) /Codeine Preparations | Tylex | Positive for Opiates (OPI/MOR) |
| Hydroxyzine Hydrochloride | Ucerax | Non-reactive |
| Tramadol | Ultram | Non-reactive |
| Hydrocortisone | Uniroid-HC | Non-reactive |
| Chlorhexidine Gluconate | Unisept | Non-reactive |
| Chlorhexidine Gluconate | Uriflex | Non-reactive |
| Chlorhexidine Gluconate | Uro-Trainier | Non-reactive |
| Norethindrone | Utovian | Non-reactive |
| Diazepam | Valclair | Positive for Benzodiazepines (BZO) |
| Diazepam | Valium | Positive for Benzodiazepines (BZO) |
| Salbutamol | Venotlin | Non-reactive |
| Beclometasone | Ventide | Non-reactive |
| Salbutamol | Ventide | Non-reactive |
| Salbutamol | Ventodisks | Non-reactive |
| Doxycycline | Vibramycin | Non-reactive |
| Doxycycline | Vibramycin- D | Non-reactive |
| l-Methamphetamine HCL | Vick's Inhaler | Positive for Methamphetamine (mAMP) |
| Hydrocodone | Vicodin | Positive for Opiates (OPI/MOR) and/or Oxycodone (OXY) |
| Hydrocortisone | Vioform-HC | Non-reactive |
| nterferon Alfa | Viraferon | Non-reactive |
| Testosterone | Virmone | Non-reactive |
| Salbutamol | Volmax | Non-reactive |
| Diclofenac Sodium | Volraman | Non-reactive |
| Diclofenac Sodium | Volsaid | Non-reactive |
| Diclofenac Sodium | Voltarol | Non-reactive |
| Bupropion | Wellbutrin | Non-reactive |
| Chloral Hydrate | Welldorm | Non-reactive |
| Interferon Alfa | Wellferon | Non-reactive |
| Alprazolam | Xanax | Positive for Benzodiazepines (BZO) |
| Doxepin | Xepin | Positive for Tricyclic Antidepressants (TCA) |
| Methylenedioxymethamphetamine (MDMA) | XTC (slang) | Positive for Methamphetamine (mAMP) and Ecstasy (MDMA) |
| Hydrocortisone | Xyloproct | Non-reactive |
| Tramadol | Zamadol | Non-reactive |
| Ranitidine | Zantac | Positive for Methamphetamine (mAMP) Urinary metabolite(s) only; parent compound is non-reactive |
| Lisinopril HCl | Zestoretic | Non-reactive |
| Lisinopril HCl | Zestril | Non-reactive |
| Metronidazole | Zidoval | Non-reactive |
| Erythromycin | Zineryl | Non-reactive |
| Cimetidine | Zita | Non-reactive |
| Sertraline | Zoloft | Potential Positive for Benzodiazepines (BZO) |
| Beclometasone | Zonivent | Non-reactive |