



# Training Program

[www.nexscreen.com](http://www.nexscreen.com)

Toll-free: 1.888.956.8989

[info@nexscreen.com](mailto:info@nexscreen.com)

# NexScreen™ Cup



- ✓ The NexScreen™ Cup is the latest generation of the best onsite drug testing product on the market.
- ✓ This training program will outline, step by step, how you can confidently administer and interpret this simple test.

# You Will Learn



- ✓ When to break the seal on the NexScreen™ Cup pouch.
- ✓ How to instruct the person who is being tested.
- ✓ The minimum urine level required.
- ✓ What to do when the Donor completes the test.
- ✓ How to read the temperature strip.
- ✓ The purpose of the Security Seal.
- ✓ How to read the test results.
- ✓ When to send the NexScreen™ Cup for confirmation.

# Quiz & Award

Once you complete the NexScreen™ Cup training, you may take a simple multiple choice quiz to ensure that you understand how to administer the test and read the results. You will be awarded a certificate upon successful completion of the NexScreen™ Cup training program.



# Instructions

1. Ensure that the NexScreen™ Cup is at room temperature (59°-83°F).
2. Open the pouch in Donor's presence.



# Instructions (continued)

3. Ask the Donor to urinate directly into the NexScreen™ Cup and return it immediately.
4. Twist on the lid, ensuring that it is not cross-threaded. Tighten the lid completely, turning it as much as possible. This prevents leakage.



# Instructions (continued)

5. Write down the time.
6. Verify that there is enough specimen (above the 30 ml minimum level line).
7. Verify that the specimen is near body temperature (90°-100°F). Look for a green indicator on the temperature strip.



# Instructions (continued)

8. Stick the security seal over the lid and down each side of the cup. Have the Donor initial the seal to attest that it is their specimen.
9. Remove the tear-off label.





# Instructions (continued)

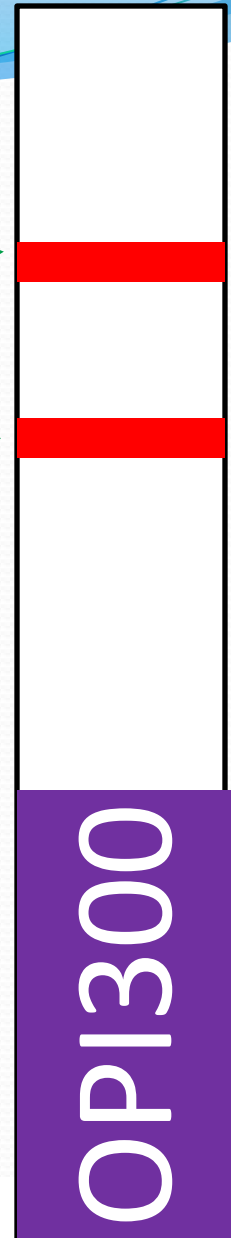
10. Read the results between 5-10 minutes.

- A legend is printed on the cup to assist you with reading the results.
- Any evidence of a line is a line, no matter how faint.



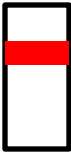

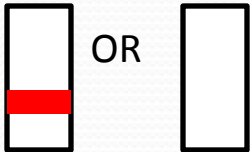
# How to Read the Results

- Test Results show as the presence or absence of red/pink lines on each strip.
- **Any** evidence of a line is a line, no matter how faint.
- Each strip has 2 possible lines:
  - 1) An upper line, called the **Control Line**
    - Its presence means that the test is working.
    - Its absence means that the test is invalid.
  - 2) A lower line, called the **Test Line**
    - Its presence means that the test is Negative for that drug.
    - Its absence means that the test is Positive, and should be sent to a lab for confirmation.



# How to Read the Results

- Read each strip independently of the other strips.
- Read **any** evidence of a line as a line, no matter how faint.

Image of strip:	What you see on a strip:	Result:	What to do:
	Control Line: present Test Line: absent	Positive	Record result, send cup to lab for confirmation testing
	Control Line: present Test Line: present	Negative	Record result and discard cup
	Control Line: absent Test Line: present or absent	Invalid	Record result, discard cup, and RETEST with another NexScreen™ Cup

# Training Complete

- Congratulations! You have completed the NexScreen™ Cup training presentation.
- Please take the NexScreen™ Cup Quiz that follows, and receive your certificate!

# Thank you!



Successfully complete this exam, and NexScreen<sup>™</sup> will e-mail you a handsome certificate showing you as an authorized NexScreen<sup>™</sup> Cup collector. **Please PRINT Clearly. Send your completed exam by fax to 1-888-957-8989, or by e-mail to info@nexscreen.com.**

Name: \_\_\_\_\_ Date: \_\_\_\_\_  
 e-mail: \_\_\_\_\_ Title: \_\_\_\_\_  
 Company: \_\_\_\_\_ Tel: \_\_\_\_\_  
 Street: \_\_\_\_\_ Fax: \_\_\_\_\_  
 City: \_\_\_\_\_ State: \_\_\_\_\_  
 Zip: \_\_\_\_\_ Company for whom you are testing: \_\_\_\_\_

<p>1. When should the test device be removed from the foil pouch?</p> <ol style="list-style-type: none"> <li>5 minutes prior to beginning the test</li> <li>Before the subject enters the room</li> <li>Immediately prior to beginning the test</li> </ol> <p>2. What is the minimum volume of urine needed to perform the test?</p> <ol style="list-style-type: none"> <li>10 ml</li> <li>20 ml</li> <li>30 ml</li> </ol> <p>3. What should you look for on the temperature strip?</p> <ol style="list-style-type: none"> <li>Green color formation</li> <li>Blue color formation</li> <li>Orange color formation</li> </ol>	<p>4. Why do you have the donor sign the security seal when placing it over the cup?</p> <ol style="list-style-type: none"> <li>To get their signature for your files</li> <li>To attest that it is their urine specimen in the cup</li> <li>Not important just a formality</li> </ol> <p>5. How long should you wait after the NexScreen<sup>™</sup> Cup test has started before reading the results?</p> <ol style="list-style-type: none"> <li>2 minutes</li> <li>3 minutes</li> <li>5 minutes</li> </ol> <p>6. Two colored lines on a drug strip indicate which result:</p> <ol style="list-style-type: none"> <li>Negative</li> <li>Positive – send to lab for confirmation</li> <li>Invalid</li> </ol>	<p>7. No lines on a drug strip indicates which result:</p> <ol style="list-style-type: none"> <li>Negative</li> <li>Positive – send to lab for confirmation</li> <li>Invalid</li> </ol> <p>8. Only one upper line on a drug strip indicates which result:</p> <ol style="list-style-type: none"> <li>Negative</li> <li>Positive – send to lab for confirmation</li> <li>Invalid</li> </ol> <p>9. Only one lower line on a drug strip indicates which result:</p> <ol style="list-style-type: none"> <li>Negative</li> <li>Positive – send to lab for confirmation</li> <li>Invalid</li> </ol>
-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

**Send your completed exam by fax to 1-888-957-8989, or by e-mail to info@nexscreen.com.**

NexScreen LLC - P.O. Box 17205 - Denver, CO 80217-0205

Tel: 1.888.956.8989 - Fax: 1.888.957.8989 – www.nexscreen.com